

Purpose of use

Solid Gutta Percha used for root canal filling

Preparation before use

1. Before using the product, check the expiration date and make sure there is no dirt, dust, or any foreign substance. Do not use if the Gutta Percha is too hard and not elastic.
2. Read and fully understand the manufacturer's instructions and study the step-by-step guide thoroughly.
3. Prepare all accessories and instruments required prior to the procedure.

Directions for use

In order to fill a root canal, a dental professional must follow the following steps.

1. Select a Gutta Percha master cone that fits the apex and check the working length before filling the root canal. Cut off the useless cone tip before inserting the Gutta Percha into the root canal.
2. When the Gutta Percha point is placed inside the root canal, take an x-ray to confirm that the Gutta Percha point is located 1mm away from the apex and perfectly fits the root canal
3. Remove the Gutta Percha point from the root canal and dry with a paper point. Choose a file that is one size smaller than the one used to perforate the root canal. (Also a professional endodontic instrument can be used.) Apply the sealer to the file and spread the inside of the canal. Finish up by applying the sealer to Gutta Percha point tip around 1mm and inserting it into the root canal.
4. This procedure is the most fundamental method. However, a dental professional may consider using more Gutta Percha points or perform the procedure in a different way based on the patient's symptoms.

- ◆ Gutta Percha points are for a single use. Do not re-use them.

Maintenance after use

Gutta Percha points that are not used after the procedure must be stored in a container at room temperature to prevent contamination


Handling & Storage

1. Store at room temperature.
2. Keep away from flammable and combustible materials and heat.
3. Do not toss or throw the products while conveying.
4. Store separate from other products of different sizes and types.

Precautions for use

- ◆ General cautions and warnings
 1. The product must be used only by dental professionals. The directions for use must be followed carefully.
 2. Do not use on patients who are inflamed by or allergic to the product.
 3. This product is for external use only and must be prescribed for internal use.
 4. Cover the patient's respiratory tract with a rubber dam before the procedure.
 5. Close the lid tightly after use.
- ◆ Cautions when using
 1. Check any secondary infection inside the root canal before placing the Gutta Percha Point.
 2. Use the same size and thickness of file and Gutta Percha point.
 3. Take an X-Ray after the procedure to ensure complete filling of the canal.

Symbols

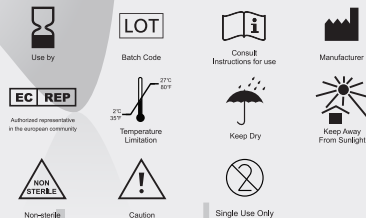

Shelf life

4 years from date of manufacture

Warranty and Limitation of Liability

Warranty

DiaDent Cambodia will replace product that is proven to be defective. DiaDent Cambodia does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

European Representative

DiaDent Europe B.V.
Antennestraat 70, 1322AS
Almere, the Netherlands
Tel: 00 31 (0)36 549 8607 FAX: 00 31 (0)36 536 7317