DIA PEN™

Cordless Warm Compaction Device

INSTRUCTION MANUAL

CORDLESS WARM VERTICAL COMPACTION DEVICE

DiaDent Network

DiaDent Group International (Korea & International)
19-9 Songjung-Dong Heungduk-Gu Cheongju-si
Choongchong Buk Do, Rep. Of Korea
Tel: 82-43-266-2315 Fax: 82-43-262-8658
E-mail: diadent@chollian.net

DiaDent Group International Inc. (Canada & USA)
11-3871 North Fraser Way, Burnaby B.C, Canada V5J 5G6
Tel: 1-604-451-8851 Fax: 1-604-451-8865
E-mail: diadent@diadent.com www.diadent.com

DiaDent Europe B.V (Europe)
Antennestraat 70, 1322 AS Almere, The Netherlands
Tel: 31-36-549-8607 Fax: 31-36-536-7317
E-mail: diadent@diadent-europe.com www.diadent-europe.com

	Page
English	05
Français	19
Deutsch	33
한국어	47

Table of Contents

1. Introduction
2. Safety Instructions
3. Package Contents
4. Product Features
 - Handpiece
 - Charger
5. Operation
 - Charging
 - Placing the silicone cover
 - Inserting the pen tip
 - Powering On
 - Setting tip size preference
 - Setting temperature preference
 - Operating
 - Powering Off
6. Cleaning and Maintenance
 - Cleaning the handpiece
 - Cleaning the charger
 - Cleaning the pen tips
 - Cleaning the silicone covers
7. Disposal
 - Handpiece and charger disposal
 - Battery pack disposal
 - Tips disposal
8. Device Item Codes
9. Warranty
10. Troubleshooting
11. Technical Specifications
12. Symbol Definitions

Thank you for purchasing DiaDent's Cordless Warm Vertical Compaction Device – Dia-Pen. Dia-Pen supports warm vertical compaction and has proven to be effective as an integral part of downpack obturation system. Dia-Pen supports the following obturation techniques and features.

Supported Techniques:

- Softening the root canal filling material (Gutta Percha Points) during root canal treatment
- Spreading the root canal filling material (Gutta Percha Points) laterally and vertically
- Cutting the root canal filling material (Gutta Percha Points)
- Compacting the root canal filling material (Gutta Percha Points) using heat

Features:

- Excellent result with warm and vertical compaction for all canals including lateral canals. Dia-Pen offers reliable results with tightly sealed root canals.
- Cordless, ergonomic and lightweight design
- Quick heating tip reaches high level of temperature within one (1) second to save treatment time
- Color-coded pen tips are available in various sizes (#XF, F, FM, M, ML) for easy identification
- Three temperature settings (High, Medium, Low)
- Simple operation


This manual provides detailed operation and setup instructions. Please read the manual carefully before operating the device. Please keep this manual in a readily accessible place for quick and easy reference. The current manual and further information can be downloaded from www.diagun.co.kr

To reduce the risk of injury, always follow manufacturer's cautions listed below as well as other warnings and cautions throughout this manual.

Caution(s) indicates information which, if not followed properly, could result in serious injury to the patient and/or the operator and damage to the device.

1. This equipment fully observes to the regulations of Electromagnetic emissions and immunity standards of EN60601-1-2.
2. This device should be used exclusively by trained and qualified dentists and endodontists only. If the device is used by any other users that are not specified in this manual, the manufacturer has no liability due to injury from the incorrect usage.
3. Practice the technique of this device before using on patients.
4. Do not use this device on patients with pacemakers.
5. To prevent possible explosion, do not use this device in the presence of flammable anesthetics.
6. To reduce the risk of electronic shock, do not try to open, disassemble or repair the handpiece, battery pack or the charger base.
7. To prevent possible injuries, do not allow patients to handle or touch this device.
8. To prevent explosion and defects, keep handpiece, battery pack and charger base in the temperature range of -10 to +40°C, air pressure from 70 to 106 kPa, and humidity from 30 to 85%.
9. Do not immerse the handpiece, battery pack or charger base in liquid of any kind.
10. The charger and power adapter should be connected to the electrical outlet compatible with the current electrical standards.
11. Use Dia-Pen's pen tips and accessories provided by DiaDent only. Use of any other brands of accessories may cause damage or loss of performance and will thus void the warranty.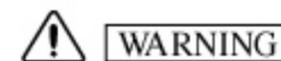

The electromagnetic waves or other medical equipments in hospitals may cause malfunction of this device.



3. Package Contents

4. Product Features

Before removing the device from the box, check the contents against the item list below to make sure you have everything. If any item is missing or damaged, contact your sales representative.

- 1 x Handpiece
- 1 x Charger
- 2 x Silicone Cover
- 1 x Power Adapter & Cord
- 1 x Pen Tips(included Endo Stop) #F
- 1 x Pen Tips(included Endo Stop) #FM
- 1 x User Manual

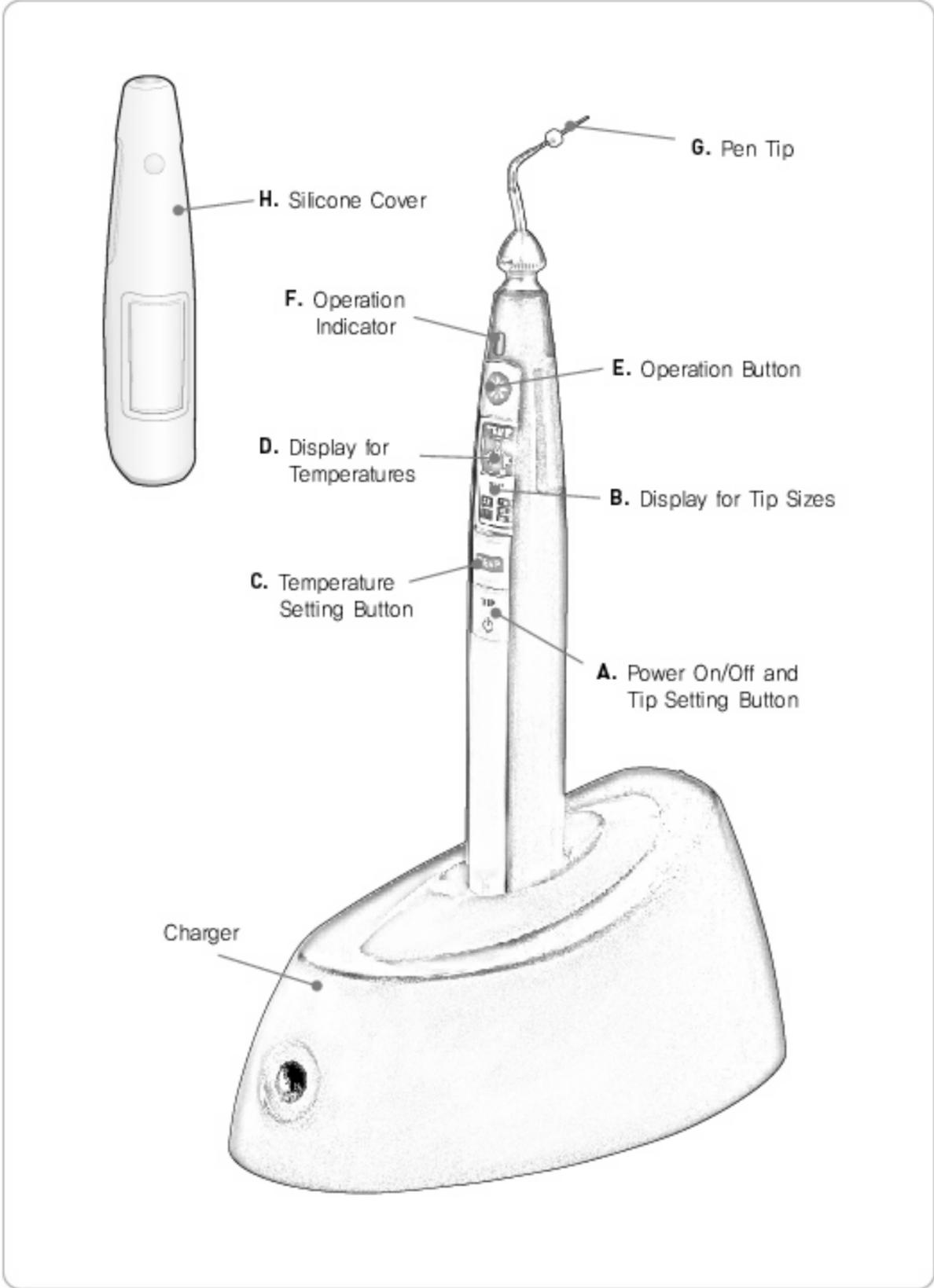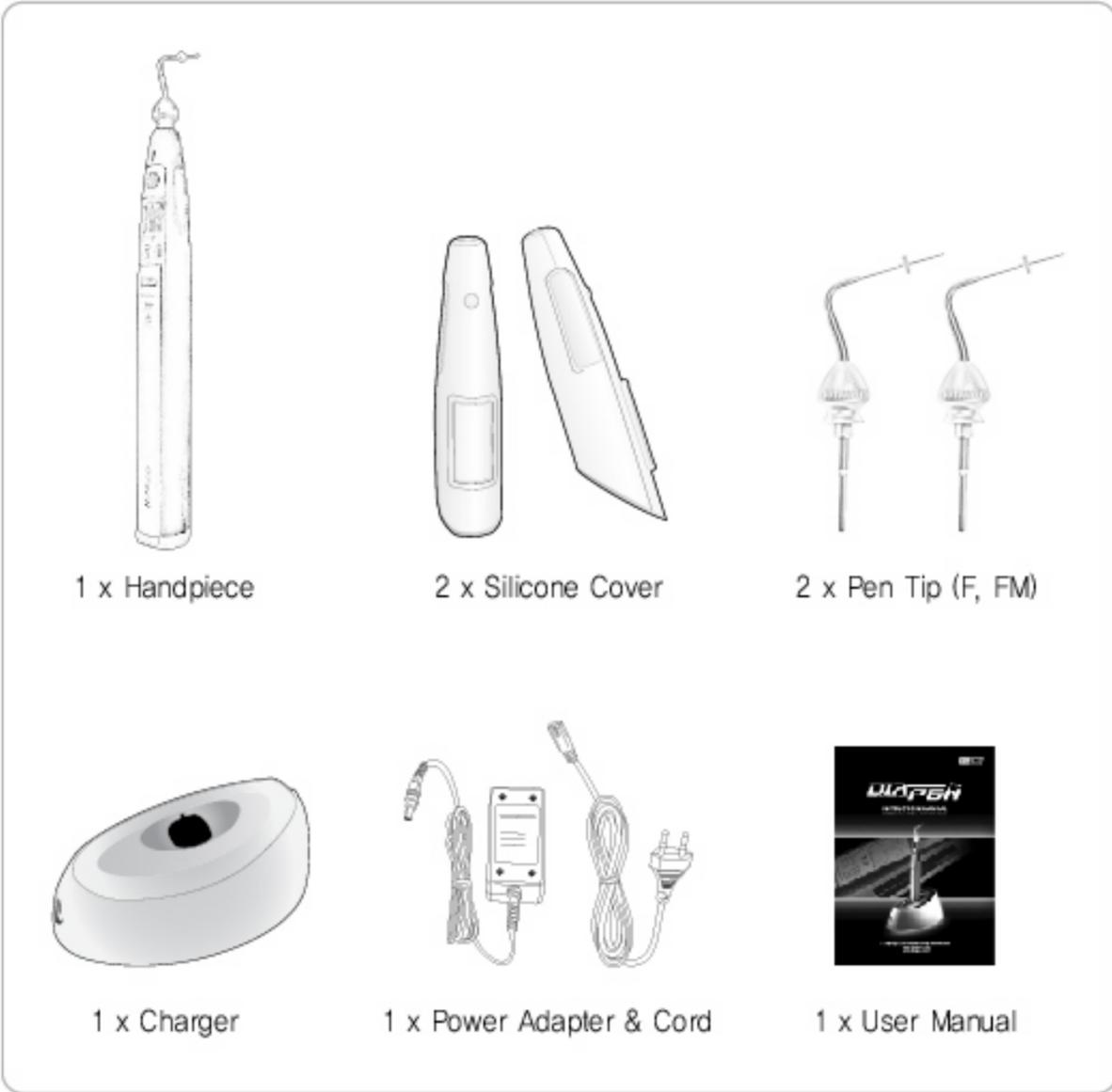HANDPIECE

A. Power On/Off and Tip Setting Button:

- Press the button and hold for one (1) second to turn the device On/Off.
- To change the Tip Size Setting, press the button again until reaching the setting you desire (#XF, F, FM, M, ML)

B. Display for Tip Sizes


B1

B2

- Bright light up left section of XF and F indicates the selection of tip size XF or F (Figure B1)
- Bright light up right section of FM, M and ML indicates the selection of tip size FM or M or ML (Figure B2)

C. Temperature Setting Button: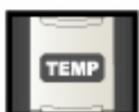

- Press TEMP button to select three temperature settings (High, Medium, Low).

D. Display for Temperatures

D1

D2

D3

- Low Temperature Mode (170°C): Indicator lights up above the "L" (D1)
- Medium Temperature Mode (200°C): Indicator lights up above "M" (D2)
- High Temperature mode (220°C): Indicator lights up above both "L" and "M" (D3)

E. Operation button

- Press to initiate pen tip heat and release to stop pen tip heat.
- Heat the pen tip by pressing and holding the Operation Button. When the tip has reached the selected temperature, a beep will sound to signal.

Pen tip temperature will cool down when the Operation Button is released. The tip will heat whenever the Operation Button is pressed; however, to prevent possible injury, the tip will not stay activated for more than two seconds.

F. Operation Indicator

F1

F2

F3

- Indicator lights when the tip is heating up and/or cooling down
- Green Light: Tip is placed in the handpiece properly and the temperature is below 100°C. (F1)
- Red Light: Tip is activated. Handpiece beeps to signal it has reached the selected temperature. (F2)
- Orange Light: Tip is cooling down for five (5) seconds. (F3)

G. Pen Tips

- Connect the tip tightly to handpiece. 5 color-coded pen tips are available.

Size	Color code	Taper/Size
#XF	Green	.025/#45
#F	White	.025/#50
#FM	Yellow	.045/#50
#M	Red	.060/#50
#ML	Blue	.080/#50

H. Silicone Cover

- Soft silicone cover can protect the handpiece.

I. Rechargeable Battery Pack

1. Before using the handpiece, check that charging is complete and the handpiece is ready for use.
2. Make sure pen tips and silicone cover are clean and undamaged before each use. Clean and disinfect pen tips and silicone cover using an autoclave before each treatment.
3. Clean, remove debris from tips and silicone covers and autoclave it as soon as possible after each use.
4. Use only pen tips and other accessories supplied with Dia-Pen device.
5. Activated pen tips can reach the temperature of 220°C or slightly higher, avoid contact with the patient and operator. Use of rubber dam is highly recommended to prevent injury during the treatment.
6. When pen tip needs to be removed after treatment, turn the power off and wait for at least one (1) minute until the pen tip has cooled down.
7. This device does not work when tip is not placed in the handpiece and/or not connected properly.
8. Do not use the heated and activated tips in the canal space for more than two (2) seconds.
9. After resetting or replacing the battery pack, Dia-Pen will return to factory default setting (Sizes: XF, F and Temperature: Medium).

CHARGER

- A. Charge handpiece battery pack when handpiece is placed in the charger and the charger is plugged in.
- B. Flashing "Red" : Battery pack is discharged. Place the handpiece in the charger as soon as this happens.
- C. Flashing "Green" : Battery pack is fully charged.
- D. After each use, always place the handpiece back in the plugged-in charger to recharge. This will ensure a longer lifetime of the battery pack.

CAUTION

1. For proper operation, the charger must be installed and used in a dry environment. If the charger becomes wet, unplug and dry it immediately.
2. Check the required voltage before connecting the charger to the power adapter. Failure to do so could damage the charger and/or the handpiece.
3. Place the charger on a level and safe surface.
4. Use only the power adapter that is supplied by DiaDent.
5. Place the handpiece vertically in the charger and check that the handpiece end cap contacts the charger contacts properly.
6. Do not drop or hit the battery pack as this may result in the exposure of corrosive chemicals the battery pack.

5. Operation

A. Charging

- Before use, check that the charging is completed.
- If the Operation Indicator on the handpiece blinks orange light, it needs to be charged. When the handpiece is discharged, place it in the charger which is connected to the power adapter.
- It is recommended that the battery be fully charged continuously for three (3) hours before using Dia-Pen for the first time.

B. Placing the silicone cover

- Place the silicone cover in front of the device and the tip before use.
- Always steam autoclave (132°C) the silicone cover before and/or after each use.

C. Inserting the pen tip

- Use DiaPen's tips only.
- Do not activate the handpiece while inserting the tip.
- Insert and push in the tip that has been sterilized into the handpiece securely.
- Select appropriate pen tip to perform the precise operation.
- 5 Color-coded pen tips with endo stops - XF, F, FM, M, ML are available.

<Inserting a tip>


- Pen tip is extremely hot during and after the operation, avoid direct contact with patient and operators to prevent severe injuries.
- Handpiece does not work when tip is not connected properly.

D. Powering On

- Press and hold the power button for one (1) second to turn the device on.

E. Setting Tip Size preference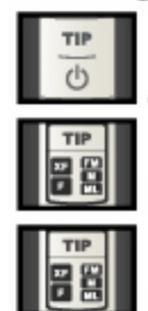

- Press the tip setting button to choose tip size. (a)
- To select tip sizes XF or F, press the tip setting button once and the left section of XF and F on display will start flashing. (b)
- To choose tip sizes FM or M or ML, press the tip setting button twice and the right section of FM, M and ML on display will start flashing. (c)
- When wrong tip size is placed, pen tip may encounter problem and not reach the desired temperature.


F. Setting Temperature preference

- Press the TEMP button to choose the desired preset temperatures (High, Medium, Low).
- Ensure the tip temperature setting is appropriate for the obturation techniques you want to use with DiaPen.
- Press the TEMP button once and indicator above the "L" will light up; temperature is selected as Low mode (170°C).
- Press the TEMP button twice and indicator above the "M" will light up; temperature is selected as Medium mode (200°C).
- Press the TEMP button three times, indicators above both "L" and "M" will light up; temperature is selected as High mode (220°C).
- Press and hold the TEMP button for three (3) seconds to reset the program if the device fails to work properly due to internal program errors.


G. Operating

Root Canal Obturation Technical Sequence

Downpack Obturating with Dia-Pen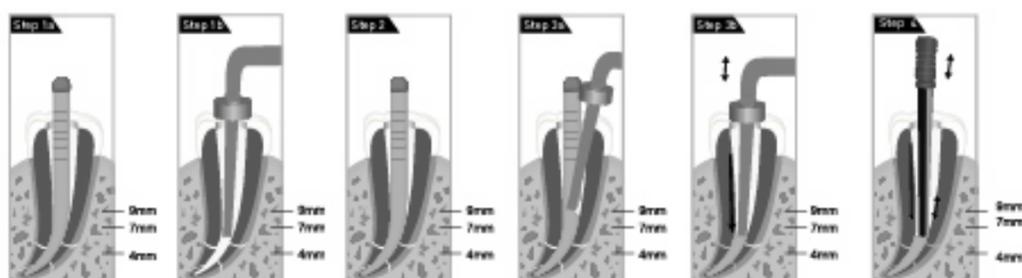

Backfill Obturating with Dia-Gun

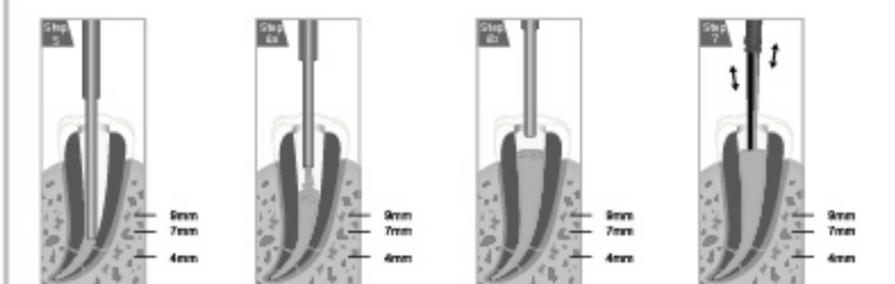- Step 1a** Shape the root canal with Endo Files and check the working length. Select the master Gutta Percha cone to be filled in the canal.
- Step 1b** Insert pen tip into the root canal to the point 4-7mm short of the working length and fit Endo Stop in the pen tip.
- Step 2** Fill the master Gutta Percha cone that is dipped in a sealer.
- Step 3a** Place pen tip in the orifice of the canal and activate the device by pressing the Operation Button. Using the activated pen tip, cut the middle of Gutta Percha cone and downpack it to the point 6-9 mm from the APEX for about 1.5 - 2 seconds only.
- Step 3b** Release the Operation Button to allow pen tip to begin cooling, while at the same time continue pushing and holding the cooled pen tip to the APEX for 8 seconds to fill the canal completely. Activate the pen tip again by pressing and holding the Operation Button for about 1 - 1.5 seconds, compact the Gutta Percha cone to the point 4-7 mm of the working length.
- Step 4** Condense the remaining material with hand plugger.
- Step 5** Using DiaGun's tip, make contact with the upper part of filled master cone in the canal, transfer heat for 3 to 5 seconds.
- Step 6a** Flow the melted Gutta Percha Obturator to the root canal.
- Step 6b** Continue flowing while withdrawing the needle from the canal.
- Step 7** Use a hand plugger to compact obturation material.

H. Powering Off

- Press and hold the power button for one (1) second to turn Dia-Pen off.
- The handpiece turns off automatically when no buttons are pressed for ten (10) minutes.

A. Cleaning the handpiece

- Remove the handpiece from the charger for cleaning. Clean the outer surface of the handpiece by wiping with a soft cloth dampened with pH neutral surface disinfection solution or alcohol.
- Do not submerge the handpiece in disinfection liquids of any kind.
- Cleaning the handpiece contacts with alcohol is recommended.

B. Cleaning the charger

- Remove the handpiece from the charger. Disconnect the charger from the power adapter and power supply. Clean the outer surface of the charger by wiping with a soft cloth dampened with pH neutral surface disinfection solution or alcohol.
- Do not submerge the charger in disinfection liquids of any kind.
- Cleaning the charger contacts with alcohol is recommended.

C. Cleaning the pen tips

- Disconnect tips from the handpiece.
- Remove all visible soil from tips and then steam sterilize the tips for a minimum of 12 minutes at 132°C or clean tips by alcohol.
- Always clean and disinfect tips with sterilization equipment before use.
- Clean Endo stops with tips using steam sterilization.

D. Cleaning silicone covers

- Remove silicone covers from the handpiece.
- Before each use, steam autoclave the silicone covers.

7. Disposal

A. Handpiece and charger disposal

- Remove battery pack before disposal. Dispose of electronic parts as electronic waste. Dispose of all other parts according to the relevant regulations concerning waste disposal.

B. Battery disposal

- Dispose the battery pack as chemical waste. Dispose in accordance with all applicable local regulations.

C. Tips disposal

- Dispose tips as medical waste. Dispose in accordance with all applicable local regulations.

Item	Codes
Dia-Pen Regular Kit	#4002-1001
Handpiece	#4002-1010
Charger	#4002-1020
Power Adapter & Cord	#4002-1030
Silicone Cover (2ea)	#4002-1040
Battery	#4002-1050
Pen Tips #XF (1ea)	#4002-1060
Pen Tips #F (1ea)	#4002-1061
Pen Tips #FM (1ea)	#4002-1062
Pen Tips #M (1ea)	#4002-1063
Pen Tips #ML (1ea)	#4002-1064
Pen Tips #F/FM (1ea each)	#4002-1065
Endo Stops (100ea)	#4002-1070

9. Warranty

- A. 1 Year Limited Warranty (excluding Silicone Cover, Pen Tips, Power Adapter & Cord)
- B. The manufacturer gives one (1) year warranty effective from the date of purchase. Any defects or errors that are due to faulty manufacturing or material will be repaired or replaced within this warranty period.
- C. To obtain warranty coverage and to return the device to the manufacturer's service facility, please contact your sales representative. Check and inform the sales representative of the reference number on the handpiece.
- D. Products returned for warranty coverage should be securely packed and shipped with postage prepaid. DiaDent does not accept shipments sent charges-collect. DiaDent will be responsible for shipping the repaired or replacement product back to you.
- E. Manufacturer disclaims liability and is not responsible for repair or replacement of product in the case of :
 - Noncompliance with this instruction manual.
 - Operating and/or handling the device without care.
 - Damage caused by natural disasters.
 - Incorrect diagnosis or poor result due to operator error.
 - Device damages resulting from self repairs or modifications will not be serviced by the manufacturer.

Problem	Action
Device will not operate	<ul style="list-style-type: none"> • Check charger and power adapter for proper operation and connection • Charge the battery pack as soon as possible • Check the plug socket
Handpiece does not work	<ul style="list-style-type: none"> • Check battery pack is fully charged • Replace damaged Pen tips • Check charger and power adapter • Replace the battery pack
Charger does not work	<ul style="list-style-type: none"> • Check that the charger light is on • Check that the power adapter is plugged into power outlet compatible with the voltage • Check to see if the charger or power adapter is damaged • If charger light is not on, please contact your representative • Ensure that the handpiece is placed firmly into the charger base • If the battery contact points of the handpiece or charger are wet, dry immediately and do not use. • Replace the battery pack • If the charger still does not work, please contact your representative
Tip does not heat	<ul style="list-style-type: none"> • Check that the battery pack is fully charged • Check that the pen tip is connected properly • Replace damaged tips • Replace damaged battery pack

Product Name	Dia-Pen	
Classification	Class II (AC/DC Adapter)	
Setting Temperature	170, 200, 220°C	
Charging time	Approx. 2 hrs 30 min	
Operation time per full charge	450 times per 2 sec	
Dimensions	Handpiece	22 x 166 x 20.1mm (W x H x D)
	Charger	156 x 66 x 88mm (W x H x D)
Weight	Handpiece	65g
	Charger	285g
	Adapter	200g
Power Supply (Adapter)	Input	100V-250V~, 50-60 Hz, 0.5A
	Output	6V  , 2.5A
Battery	Li-polymer Battery, DC 3.7V, 1200mAh	
Storage Condition	Temperature	-10°C ~ +40°C
	Humidity	30 ~ 85%
	Atmospheric Pressure	70 ~ 106 kPa
Transportation Condition	Temperature	-10°C ~ +40°C
	Humidity	30 ~ 85%
	Atmospheric Pressure	60 ~ 106 kPa

12. Symbol Definitions

Symbol	Description
	Cautions indicate which, if not observed, could result in damage to person and/or equipment
	Direct current
	CE certification
	Manufacturer
	Manufacture date
	Serial No
	EU Representative
	FRAGILE
	KEEP DRY
	USE NO HOOKS/DO NOT PUNCTURE
	UNSTABLE

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